

5.2 Installation

5.2.1 Generalities

- Install the NurseCall N46 Wall Transmitter in a dry place, away from any source of heat.



CAUTION!

Interferences

Avoid immediate proximity to other electric devices such as a television.

5.2.2

Wall installation

The NurseCall N46 Wall Transmitter can be fixed with two to five screws on a wall. Two holes at 60 mm interval are provided for direct mounting on connection boxes.

The three upper holes are used for positioning the device and cannot be reached from the installation compartment.

Two of the lower holes can be reached from outside and thus used for fixing the device.

